Porcelain Etch Gel

9,6% Hydrofluoric Acid

-  Porcelain etching gel syringes
-  Porselein etsgel syringes
-  Seringues d'Etching pour la porcelaine
-  Porzellan Ätzgel Spritzen
-  Porcelain aets gel i sprøjter

instructions for use

Danger
Gevaar
Danger
Gefahr
Fare

Polo MB Oisterwijk B.V.
Laarakkerweg 5
NL- 5061 JR Oisterwijk
www.polodent.com

Porcelain etching gel syringes

Application:

Porcelain etching gel is used for porcelain restoration i.e. Inlays, Onlays, Veneers or crowns.

 Use by date:
See overprint

Product description:

PoloDent Porcelain Etch Gel is hydrofluoric acid based and serves as **extraoral** etching for porcelain restoration.

 Batch code:
See overprint

Instructions for use:

- Apply the etching gel to the porcelain surface with a small disposable brush or disposable application tip.
- The recommended reaction time is minimum 60 sec.
- Acid residue should be completely removed after the reaction time. The surface can be steam cleaned, rinsed with plenty of water or cleaned with distilled water in an ultrasonic cleaner to remove the acid residue. The porcelain surface should be etched immediately before fitting or after try-in. Avoid contaminating the etched surfaces. If surface is contaminated, it should be briefly re-etched (approx. 10 sec). Dry thoroughly, then apply bonding agent etc. according to the manufacturer's instructions. Only intended for extra oral use. Should not be used for etching tooth structure intraorally.

H300:	Fatal if swallowed.
H310:	Fatal in contact with skin.
H330:	Fatal if inhaled.
H314:	Causes severe skin burns.
H318:	Causes serious eye damage.
P261:	Avoid breathing dust/fume/gas/mist/vapours/spray.
P262:	Do not get in eyes, on skin, or on clothing.
P280:	Wear protective gloves/protective clothing/eye protection/face protection.
P301+P310:	IF SWALLOWED: Immediately call a POISON CENTER or doctor/physician.
P301+P330:	IF SWALLOWED: Rinse mouth.
P331:	Do NOT induce vomiting.
P302+P352:	IF ON SKIN: Wash with plenty of soap and water.
P305+P351+P338:	IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P312:	Call a POISON CENTER or doctor/physician if you feel unwell.
P321:	Specific treatment (see label).
P330:	Rinse mouth.
P361:	Remove/Take off immediately all contaminated clothing.
P403:	Store in a well-ventilated place.
P233:	Keep container tightly closed.

Dispose of contents. Container is according to the local / regional / national / international regulations.

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