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PoloDent Ultrasonic Cleaner is enzyme based for the pre-cleaning and decontamination of inanimate, non-porous surfaces of hand instruments and forceps, rotary instruments, mirrors etc. Designed for use in the ultrasonic bath. Specially formulated for use in the medical & dental environment.

Directions for use: Add 30 ml \pm 1 ml of the concentrate to 970 ml of tap water in the ultrasonic bath (3% solution). Immerse the instruments ensuring that all surfaces are covered by the solution, add hollow objects in such a way that air escapes. Run the ultrasonic cleaner for a minimum of 15 minutes and according to the manufacturers instructions. After immersion, remove the instruments and rinse them thoroughly prior to terminal sterilisation or high level disinfection. Prepare a fresh solution daily or sooner if solution is very dirty. For professional use only.

Composition: Aqua, Propan-2-ol, Didecyltrimethylammonium chloride, Sodium n-lauroylsarcosinate, Subtilisin, CI 19140.

Warning:

H226 - Flammable liquid and vapour.

H315 - Causes skin irritation.

H319 - Causes serious eye irritation.

H336 - May cause drowsiness or dizziness.

P302+P350 - IF ON SKIN: Gently wash with plenty of soap and water.

P305+P351+P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P280 - Wear protective gloves/protective clothing/eye protection/face protection.

P261 - Avoid breathing dust/fume/gas/mist/vapours/spray.

P304+P340 - IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing.

Storage

P210 - Keep away from heat/sparks/open flames/hot surfaces. - No smoking.

P403+P233 - Store in a well-ventilated place. Keep container tightly closed.

P235 - Keep cool.

Contraindications

Incompatible materials: Aluminium, copper. Not recommended for plastics and rubbers.

Disposal

Do not discharge concentrate into drains or rivers.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

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