# POLO DENT<sup>®</sup>

HYGIENE

## Disinfection Wipes

18x12 CM

- en Surface Disinfectant Wipes
- nl Doekjes voor oppervlakkendesinfectie
- fr Lingettes de désinfection de surfaces
- de Desinfektionstücher für der Oberflächen
- pl Chusteczki do dezynfekcji powierzchni
- da Desinficerende overfladeservietter
- sv Vätservietter för ytdesinfektion
- no Vätservietter for overflatedesinfisering
- fi Pintadesinfointipyyhkeet

100 pieces

**MD** For the surface disinfection of non-invasive medical devices. Alcohol is not recommended for use on dental/medical instruments.

**BPR** For the disinfection of surfaces. Use biocides safely and sustainably. This product should be used in accordance with the product label.  
PT2 Formulation type: AL.

**Directions for use:** 1. Remove cap of dispenser tub, 2. Cut open air tight plastic pouch across the top of the pouch, 3. Feed the centre wipe through the hole in the cap and replace, 4. Pull wipe at a 45° angle. Using sufficient wipes for each application wipe the clean surfaces to be disinfected so that they are completely moistened and let the disinfectant take action for 60 seconds. Allow surface moisture to evaporate. For professional use only.

**Efficacy (60 secs):** Bactericidal including MRSA, VRE. Yeastocidal. Mycobactericidal. Tuberculocidal. Viruses to include: Non-enveloped viruses: Murine norovirus, Adenovirus 5. Enveloped viruses: Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus, HIV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles, Poxviridae.

**Tested to Standards:** EN 16615, EN 14885, EN 14476, EN 14348, EN 13727, EN 13624.

**Composition:** Ethanol (64-17-5) 63% w/w (630g/kg), Aqua, Perfumes.

**Danger.** H225: Highly flammable liquid and vapour. H319: Causes serious eye irritation. **First aid procedures.** P303+P361+P353: IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **Fire procedures.** P370+P378: In case of fire: Use alcohol resistant foam or carbon dioxide to extinguish. **Storage.** P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool. **Contraindications.** Incompatible materials: Copper, aluminium. Do not use on surfaces sensitive to alcohol. **Disposal.** Dispose of contents/container in accordance with local regulations. Do not discharge liquid into drains or rivers. Do not flush. Non maceratable.

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
*Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.*

## Disinfection Wipes

18x12 CM

**REF** 82 90 10

Use by: Add expiry date from refill bag

Tub refilled x  
(replace tub after 12 refills)

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