POLO | DENT[®]
HYGIENE

Disinfection Wipes AF

18x12 CM

- en Surface Disinfectant Wipes alcohol-free
nl Alcoholvrije oppervlakkenreinigings- en desinfectieoekjes
fr Lingettes de désinfection de surfaces sans alcool
de Alkoholfreie Flächenreinigungs- und Desinfektionstücher
pl Środek dezynfekujący do odkażania powierzchni bez zawartości alkoholu
da Alkoholfri overfladedesinfektionsserviet
sv Vätservetter med alkoholfritt ytdesinfektionsmedel
no Alkoholfri serviett til desinfeksjon av overflater
fi Alkoholiton pintojen desinfiointipyyhe

0%
Alcohol

100 pieces

MD

Specially formulated for use in the medical and dental environment. Contains detergent. For the surface disinfection of medical devices, including but not limited to, dental chairs, examination lights and patient transport. Not intended for use on medical and dental instruments.

BPR

For the disinfection of surfaces. Contains detergent. Use biocides safely and sustainably. This product should be used in accordance with the product label. PT2 Formulation type: AL.

Directions for use: 1. Remove cap of dispenser tub, 2. Cut open air tight plastic pouch across the top of the pouch, 3. Feed the centre wipe through the hole in the cap and replace, 4. Pull wipe at a 45° angle. Using sufficient wipes for each application wipe the clean surfaces to be disinfected so that they are completely moistened and let the disinfectant take action over 60 seconds. Allow surface moisture to dry or wipe with clean tissue. For professional use only.

Efficacy (60 secs): Bactericidal. Yeastocidal. Virucidal activity against enveloped viruses: Vaccinia virus, Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus, HIV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles, Poxviridae.

Tested to Standards: EN 16615, EN 14885, EN 14476, EN 13727, EN 13624.

Composition: Contains Aqua, Alkyl (C12-16) dimethylbenzyl ammonium chloride (ADBAC/BKC (C12-16)) (68424-85-1) 0.48% w/w (4.8g/kg), Didecylmethylammonium Chloride (7173-51-5) 0.22% w/w (2.2g/kg), Perfumes.

Warning. H412: Harmful to aquatic life with long lasting effects. **First aid procedures.** P301+P310: IF SWALLOWED: Immediately call a POISON CENTER or doctor. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. The wearing of protective gloves is recommended. UK number for poison advice: 111. Poisons Information Ireland: For information or to report a poisoning incident contact The National Poisons Information Centre (01 8092166). **Storage.** P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool. **Contraindications.** Incompatible materials: mild steel, aluminium, copper. **Disposal.** Dispose of contents/container in accordance with local regulations. P273: Avoid release to the environment. Do not discharge liquid into drains or rivers. Do not flush. Non maceratable.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

Disinfection Wipes AF

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
REF

82 90 40

18x12 CM

Use by: Add expiry date
from refill bag

Tub refilled x
(replace tub after 12 refills)

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