Maxi Disinfection Wipes

20x20 CM

REF 82 90 60

POLO DENT[®]

HYGIENE

(GB) Disinfection wipes for surfaces of non-invasive medical devices. Specially formulated for use in the medical and dental environment. Using a new wipe for each application, wipe objects to be disinfected so that they are completely moistened. Let the disinfectant take action for a minimum of 60 seconds (to achieve full efficacy leave for 5 minutes). Allow surface moisture to evaporate. For professional use only.

- **Effective spectrum:** Bactericidal (60 secs) including MRSA, VRE, Yeasticide (60 secs), Mycobactericidal (60 secs), Tuberculocidal (60 secs), Fungicidal (5 mins), Viruses to include (60 secs): Non-enveloped viruses; Murine norovirus, Adenovirus 5, Enveloped viruses; Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus (HDV), Human Immunodeficiency Virus (HIV), Influenza Virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles. Tested to: EN 14885, EN 14476, EN 14348, EN 13727, EN 13624.
- **Composition:** Ethanol, Aqua, Parfum, Linalool, Limonene. **Active:** Ethanol 63% w/w (630g/kg).
- **Danger:** H228: Flammable solid. H319: Causes serious eye irritation. P280: Wear protective gloves/protective clothing/eye protection/face protection. P303+P361+P353: IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P370+P378: In case of fire: Use alcohol resistant foam or carbon dioxide to extinguish. **Storage:** P210: Keep away from heat/sparks/open flames/hot surfaces and other ignition sources - No smoking. P403+233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool. **Contraindications:** Incompatible materials: Aluminium, copper. Do not use on surfaces sensitive to alcohol. Alcohol is not recommended for use on dental/medical instruments. **Disposal:** Do not discharge liquid into drains or rivers. Do not flush. Non maceratable. *Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.*

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Topdental (Products) Ltd
DIRECTA AB