**PoloDent LC Universal Bond** is a simple to use light cure universal adhesive. LC Universal Bond can be used with the self-etch technique, with preceding selective enamel etching or with the total-etch technique. It is suitable for moist, wet and dry surfaces. Based on innovative MDP technology LC Universal Bond allows an excellent adhesion.

#### Indications:

Adhesive for the self-etch technique, selective enamel etching or total-etch technique for:

- Direct light cure composite and compomer restorations.
- Core build-ups with light, dual and self cure composites if light cure of the adhesive is possible.
- Cementing of inlays, onlays, crowns and bridges with light, dual and self cure composite cements if light cure of the adhesive is possible.
- Repair of fractured composite restorations.

**Use PoloDent LC Universal Bond only with dual cured or self cured composites if light cure of the adhesive is guaranteed.**

#### Side Effects:

In singular cases, the material may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.

#### Incompatibility with other materials:

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the material. Neither store the material in proximity of eugenol containing products, nor let the material come into contact with materials containing eugenol.

#### Application:

- 1. Isolation** - Rubber dam is the recommended method of isolation.
- 2. Cavity Preparation** - Prepare the cavity according to the principles of the adhesive restoration technique.
- 3. Pulp Protection** - For pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.
- 4. Conditioning with etching gel (optional)** - Selective enamel etching or etching by application of the total-etch-technique can be used to improve the bond to enamel.

**Note:** Unprepared enamel surfaces must always be conditioned with etching gel.

**Etching precaution:** It is essential, that etched areas are not contaminated by anything. If contamination occurs, re-etch, rinse with water and dry as above. Avoid etching gel contact with oral soft tissues, eyes and skin. If accidental contact occurs, flush immediately with copious amounts of water.

#### a. Selective enamel etching

Apply etching gel onto the enamel surfaces and leave in place for **at least 15 seconds**. Deciduous teeth are etched correspondingly longer. Rinse for 20 seconds with water and dry it in a water-free and oil-free airstream. The etched enamel surface should have a chalky white appearance.

#### b. Total-Etch Technique

Apply etching gel onto the enamel and dentine surfaces beginning with the enamel bevels. Condition the enamel for **at least 15 seconds** and the dentine for **15 seconds**. Deciduous teeth are etched correspondingly longer. Rinse for 20 seconds with water. Dry it in a water-free and oil-free airstream, but do not desiccate. The etched enamel bevel should have a chalky white appearance.

**5. Application of PoloDent LC Universal Bond** - Dispense one or two drops LC Universal Bond into a mixing well and use immediately after dispensing. **Close bottle immediately after use.**

Apply LC Universal Bond with a brush onto the **whole** enamel and dentine surface in a thin and even layer. Keep the adhesive agitated for **at least 20 seconds**. Applying the adhesive on the tooth surface without agitation is inadequate. The material should build a homogeneous layer. Air thin gently (**at least 5-10 seconds**) to remove the volatile components and to disperse the adhesive until a glossy and uniform layer results. Then light cure with a suitable dental halogen light unit or an LED (wavelength 400-500 nm, light intensity min. 1000 mW/cm<sup>2</sup>) for **20 seconds**.

**6. Application of a composite** - Refer to manufacturer instructions for placement of the light, dual or self cure restorative material.

**Note for direct restorations:** Best results are obtained with application of a thin layer of a light cure flowable composite followed by the application of a moldable composite. Light cure each composite layer separately according to the corresponding user instructions.

#### Warnings:

- Unpolymerized material may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes.
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.

#### Storage:

Do not store above 25°C (77°F)! Avoid storage in direct sunlight. Do not use after expiration date.

#### Warranty:

DS Dental Supply GmbH warrants this product will be free from defects in material and manufacture. DS Dental Supply GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and DS Dental Supply GmbH's sole obligation shall be repair or replacement of the DS Dental Supply GmbH product.

#### Limitation of Liability:

Except where prohibited by law, DS Dental Supply GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

**Keep away from children!  
For dental use only!**

*Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.*

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