Al possible Handpiece Lubricant Spray for maintenance of all dental handpieces including high speed, low speed and air motors. Contains white mineral oil with a trace of peppermint oil. **Instructions for use:** Insert into the back of the handpiece and spray through for one to two seconds immediately after each use and before sterilisation. Wipe excess thoroughly. Important: Hold can in a vertical position when in use. Refer to handpiece maintenance instructions where applicable. Use only as directed. For professional use only. **Warning:** H222: Extremely flammable aerosol. H229: Pressurised container; may burst if heated. P211: Do not spray on an open flame or other ignition source. **Storage:** P403+P235: Store in a well ventilated place. Keep cool. P410+P412: Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122 °F. **Contra-indications:** P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. **Disposal:** P251: Do not pierce or burn, even after use. *Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.*

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REF 51 86 00

LOT

For LOT number and expiry date see can

UF: CW10-G031-P001-5P9X

Warning  
Waarschuwing  
Attention  
Achtung  
Uwaga  
Warnung  
Varning  
Advarsel  
Varoitus

Polo MB Oisterwijk BV  
Laarakkerweg 5  
5061 JR Oisterwijk, The Netherlands

650 3