Cheek Retractors

en	Cheek and lip retractors	03
nl	Wang- en liphouders	04
fr	Écarte-joues	05
de	Wangen- und Lippenhalter	06
pl	Rozwieracz jamy ustnej	07
da	Læbeholder	08
sv	Munvinkelhållare	09
no	Leppeholder	10
fi	Suunaukipitäjä	11

POLO DENT[®]

INSTRUMENTS

Product #	Description
59 32 53	Cheek Retractor Single Junior
59 32 55	Cheek Retractor Single Adult
59 32 59	Cheek Retractor Double Junior
59 32 60	Cheek Retractor Double Adult

Cheek Retractors

ENGLISH

POLO | DENT[®]
INSTRUMENTS

WARNING NOTICE: THE PRODUCT MUST BE COLD STERILIZED

Limitation on treatments: 50x

How to use: Medical device used to widen the cheek so as to have a better view of the operating field for intra-oral photography, for prophylaxis and orthodontic treatments.

Storage and transport: No special requirement.

Cleaning: Use cleaning agent that takes into account the instructions for cleaning the product. After cleaning, remove cleanser thoroughly under running water. When disinfecting in thermodisinfectors do not exceed 93 °C.

Disinfection: Use disinfecting solutions that take into account the instructions for disinfecting the product, remove cleanser thoroughly under running water. When disinfecting in thermodisinfectors do not exceed 93 °C.

Maintenance: No special requirement.

Packaging: Standardized packing material for sterilisation (recommendation).

Sterilisation: Autoclavable vapor sterilisation at 121 °C for 15 minutes.

Inspection and function test: Check the possible damages and possible distortion.

Storage: No special requirement.

The above-mentioned instructions have been validated as SUITABLE for preparation of medical device and its re-use by the medical device manufacturer.

It is the reprocessor's responsibility that the actual reprocessing achieves the required results with the equipment and materials applied and personnel involved in the reprocessing.

Normally, validation and routine controls are necessary.

Furthermore, the reprocessor should evaluate any deviation from the provided instructions regarding efficiency and possible adverse consequences.

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