



POLO | DENT[®]
POLISHING

Zircon Prophy Paste

CLEANING & POLISHING



Product Information

Issued 05/2023



Polo MB Oisterwijk BV
Laarakkerweg 5
5061 JR Oisterwijk, The Netherlands



Prisman GmbH
Otto-Hahn-Ring 6-8
64653 Lorsch, Germany
+49 6251 8669800

Zircon Propy Paste

CLEANING & POLISHING

POLO DENT®
POLISHING

Characteristics


- Effectively removes plaque and discoloration for professional tooth cleaning and prophylaxis.
- Optimized cleaning and abrasion.
- Pleasantly fresh taste of mint.
- Optimized thixotropy for perfect application.
- The paste does not spatter and does not dry out.

Areas of application

Polishing paste for professional tooth cleaning based on zirconium silicate.

Application

- With one dosage of Propy Paste a group of 2-3 teeth can be treated (cleaning and polishing in one step).
- If there are still residues on the tooth surface, they should be treated with a fresh amount of the propy paste.
- In order to avoid splattering of the material, you should work at a low speed.
- For patients with exposed dentin, the following procedure is recommended: First treat the enamel areas with a fresh amount of the propy paste and then the dentin areas.

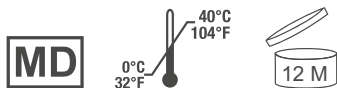

Chemical-physical data

Density at 20°C: 1,3 g/cm³
Optics: cream-colored
Viscosity: thixotropic gel with dispersed abrasive particles

