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vs.46.5272

POLO|DENT[®]

Probiotic Suction Cleaner CONCENTRATE

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en Probiotic cleaner for suction systems

PoloDent Probiotic Suction Cleaner is a concentrated and intensive working probiotic cleaning liquid for saliva suction installations, amalgam separators and sinks in dental practices. Removes efficiently all pollutions and keeps the reservoir and tubes clear. Prohibits the formation of biofilm and reduces the risk on harmful germs in the installation. Prohibits the formation of unpleasant smells. It has a long lasting effect.

Composition according to the E.C. guidelines:

<5% fosfonates, polycarboxylates; enzymes; Bacillus ferment; PHENOXYETHANOL.

Directions for use:

- PoloDent Probiotic Suction Cleaner is used as a 3% solution. Add 30 ml concentrate to 1 litre water and suck through.
- Using frequency: twice a day, before the first and after the last use of the installation.
- Shake before use.
- Empty the rinse reservoir according to the actual regulation.
- No foaming agents must enter the suction system or thespittoon bowl.

Remark:

P102 – Keep out of reach of children.

P101 – If medical advice is needed, have product container or label at hand.

Special remarks:

Keep the product at a temperature between 10°C and 45°C; shield from direct sunlight. Don't mix the product with other products like disinfectants. Diluted product has to be used within 5 days, left-overs can be removed through the drain. There are no special safety measures required for this product.

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Composition conforme aux directives C.E.:

<5% fosphonates, polycarboxylates; enzymes; Bacillus ferment; PHENOXYETHANOL.

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<5% Phosphonate, Polycarboxylate; Enzyme; Bacillus ferment; PHENOXYETHANOL.

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